



S8vExp: Exporting Files

1. Put your USB stick in the drive.
2. Press 'File' on the keyboard (bottom left, in blue).
3. Use the trackball to navigate to the patient whose images and videos you wish to export. Press 'Set' twice to enter into that patient's record, and 'Set' twice again to open the folder of the examination you wish to export.
4. You can now individually click once to highlight the files you want, or 'All Select' to highlight all of them at once.
5. If you are happy with the export format (currently on .jpg for images and .avi for cine loops by default), simply 'Export.'
6. Once your files have exported, click 'Exit.'